

Advanced Dental Designs, Inc. 9880 Indiana Avenue, #4 • Riverside, CA 92503 • USA

USA & Canada: 800.232.2849 International: +001 951.579.4535

www.ad2usa.com

((

Magnetic Mounting System Instructions Item #MP280000

1



Position the magnetic base over the frame, noting the position of the index holes.

2



Place the magnetic base on the frame.

3



Position the allen head screws (supplied) on the frame so they can be tightened.

4



Tighten the allen screws into the magnetic base until snug.

5



Position the magnet and retainer cup (supplied) over the magnetic base.

6



Tighten the mounting knob into the magnet retainer cup until snug, ensuring that the surface of the magnet sits flush with the magnetic base.

7



Repeat these steps to install the remaining magnetic base on the lower frame until completed (shown above).