

Advanced Dental Designs, Inc. 9880 Indiana Avenue, #4 • Riverside, CA 92503 • USA

USA & Canada: 800.232.2849

International: +001 951.579.4535

www.ad2usa.com

 ϵ

Mounting Plate Spacer Assembly Instructions Item #AR240079



The AD2 Mounting Plate Spacer as it appears when delivered to the customer.

3

5



Remove the mounting knob and allen screws from the plate spacer.



Thread the mounting knob supplied with the plate spacer thru the lower articulator frame and spacer plate.



Remove the lower mounting plate knob from the articulator by pulling down on it while turning counter clockwise.



Attach the plate spacer to the lower articulator frame with the two allen screws.



The installation is now complete and should look like the picture above.

6