

## Bite Fork Stabilizer Instructions

Item #TP400027

1



Press Accu-Bites on top side of bite fork and soften in hot water (125°F, 40° C) for 60 seconds.

2




Insert bite fork and obtain maxillary imprint in Accu-Bites. When finished, remove bite fork and cool with water.

3



Dry the bottom of the bite fork with a towel or compressed air.

4

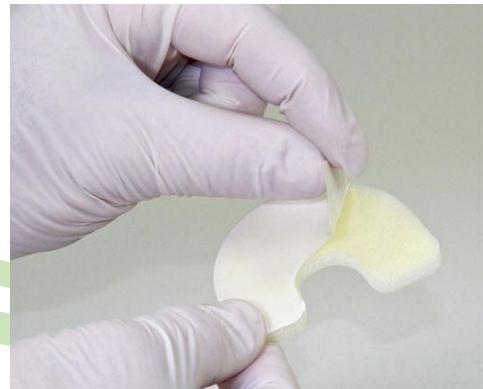

Remove the backing from the adhesive on the stabilizer

5



Place the bite fork stabilizer on the bite fork and press into place.

6



Place the bite fork in the mouth and have the patient close into the foam. Once the bite fork is stable, perform the facebow procedure.